

# Jalosome<sup>®</sup> Soothing Gel

Effective for chemo and radiotherapy skin lesions



## PRODUCT DESCRIPTION:

Vesicular suspension in hydrophilic gel of medium viscosity indicated for the treatment of skin lesions due to radiotherapy and for the reduction of associated pain.

**JALOSOME<sup>®</sup> Soothing gel** is made up of liposomes, vesicular micro-structures which, facilitating the penetration of the stratum corneum by the various components, allow a combined action, favoring the regenerative process of the skin and the control of painful symptoms.

The Jalosome line also includes JALOSOME<sup>®</sup> Hydrating Cream, a medical device, particularly indicated in the tissue repair processes.

## INDICATION FOR USE:

- Active in the treatment of skin lesions.
  - Indicated for radiodermatitis, dermatitis, skin inflammations in general, erythematous lesions of the skin, included painful lesions.
- Recommended for all body areas.

## MECHANISM OF ACTION:

**JALOSOME<sup>®</sup> Soothing gel** consists of seven components, each with a specific function within the formulation.

In particular, **JALOSOME<sup>®</sup> Soothing gel** contains hyaluronic acid gel at high and low molecular weight, in which vesicular micro-formations, essentially consisting of double layers of phosphatidylcholine, are dispersed. Their function is to convey the other components through the superficial layers of the skin and, specifically, through the stratum corneum, which constitutes the main barrier to skin permeation.

Ingredients	Properties
Hyaluronic Acid	Protects the skin from the aggression of external elements
Allantoin	Promotes the physiological cell renewal process
N-Acetylcarnitine	Creates the conditions for the physiological process of cell renewal and is useful for soothing the feeling of pain

**COMPOSITION:**

Water, L-Acetyl-L-Carnitine hydrochloride, Allantoin, Glycerin, hydrogenated phosphatidylcholine, Sodium Hyaluronate (1700 Kda), Sodium Hyaluronate (200-400 Kda), Phenoxyethanol

**CLASSIFICATION:**

Class IIa Medical Device.

**POSOLGY AND METHOD OF ADMINISTRATION:**

Each gel supply from the airless bottle - corresponding to 0.2g of gel - is sufficient to treat 16 cm<sup>2</sup> of skin. Apply the needed quantity depending on the extent of the lesion and massage gently in order to spread the product evenly; after a few minutes no residue will be present. Apply twice a day until the skin lesion is resolved, unless otherwise prescribed.

**CONTRAINDICATIONS:**

There are no known product-specific contraindications.

**SPECIAL WARNINGS AND PRECAUTIONS:**

If an allergy to the components is known, avoid using the product. In case of sensitization, stop the treatment and inform your doctor. After use, do not disperse into the environment. Do not swallow.

**INTERACTIONS WITH DRUGS AND OTHER FORMS OF INTERACTION:**

No known interactions.

**SIDE EFFECTS:**

No known side effects. Do not use in case of hypersensitivity or individual allergy to the components; in case of hypersensitivity to one of the components, discontinue administration and inform your doctor.

**INCOMPATIBILITY:**

No known incompatibilities. For a better efficacy of the product, we do not recommend the use of other moisturizing solutions before, in conjunction with or after the use of JALOSOME® Soothing gel.

**EXPIRY PERIOD:**

The product expires in 35 months from the date of production in unopened packaging and stored correctly. Once opened, the single bottle must be used within 7 days of opening.

**SPECIAL PRECAUTIONS FOR STORAGE:**

Store in a cool and dry place, away from sources of heat, light and humidity.

Do not use the product if it is damaged or not perfectly intact when it is first opened.

Do not use the product after the expiry date indicated on the package.



CODE:	DESCRIPTION:	SALES UNITS:
95810	JALOSOME® Soothing gel 20 airless bottles 10 ml	1 carton: 8 boxes
95810F	4 airless bottles 10 ml	1 carton: 28 boxes

Certified

**B Corp Certification**

It means being part of a global movement of companies that meet high standards of social and environmental impact in favor of an inclusive, equitable and regenerative socioeconomic model.

A constant commitment to the wellbeing of people, customers, communities, and the environment.

**Welcare Industries S.p.A.**

Operational site: via dei Falegnami, 7 - 05018 Orvieto (TR)  
Commercial office: Regus Building, piazzale Biancamano, 8 - 20121 Milano  
Commercial office: EUR - Spaces, viale Luca Gaurico, 91/93 - 00143 Roma  
Registered office: via San Giovanni sul Muro, 18 - 20121 Milano

[www.welcareindustries.com](http://www.welcareindustries.com)

Revision 01 2023/10

**WELCARE**  
Protecting People - Caring for Nature